HYAJOINT Plus™

Synovial Fluid Supplement

Contents:

Active ingredient: Sodium Hyaluronate 60 mg
Excipients: Disodium Phosphate Dodecahydrate, Sodium Dihydrogen Phosphate Dihydrate, Sodium Chloride, Sterile Water for Injection q.s. to 3.0 mL. pH 6,8 - 7,8

Description:

HYAJOINT Plus Synovial Fluid Supplement is a product supplied in a glass syringe with hyaluronic acid. The product is a clear, lucid, viscous, sterile and nonpyrogenic solution. The hyaluronic acid stabilized is extracted from microorganism. The product is for single use only. Hyaluronic acid is a natural polysaccharide which occurs as an important structural element of skin, subcutaneous and connective tissues, such as synovial fluid. Hyaluronic acid is identical in all living organisms and highly biocompatible. Each one mL of the product contains 20 mg of sodium hyaluronate dissolved in physiology saline.

Package: 3.0 mL per syringe

Indication:

Function of the product is indicated for treatment of pain in osteoarthritis (OA) joint.

Usage:

The product is used 1 syringe per treatment for adult into the osteoarthritis (OA) joint. The product is used for injecting to joint.

The dosage and injection time is according to symptom of patient. Anticipated duration of efficacy is at least 6 months after the product treatment. The product is used with a 21-23G needle by injecting the material into articular cavity of joint.

Therapeutic efficacy:

The product can counterbalance the mechanical stress transmitted from outside.

It is a barrier protecting the joint. The product can protect synovial tissues and promote healing of articular cartilage.

Contraindications:

- Do not administer to patients with known hypersensitivity to hyaluronate preparations.
- Intra-articular injections are contraindicated in cases of past and present infections or skin diseases in the area of the injection site.

Warning:

The product is intended for use as intra-articular implant. Don't re-sterilize the product or mix with other products.

Injections into blood vessels and surrounding tissues should be avoided. Doctor should take care of patients who ever got allergy about other drugs and who have liver disease.

If the package is damaged, do not use. If re-using, it may cause infection even seriously inflammatory response.

Usage for old men

Because the organ functions of old men are weak, doctors should pay attention to them.

Usage for mother to be and lactation

If anticipated efficacy of treatments is better than adverse events, the product could be intended to use. During the treatment period, lactation mothers should stop to suckle.

Usage for children

The safety of use for children is not sure; therefore, the doctor should pay attention when treating.

Anticipated side-effects:

The major side-effects are pain, swelling and stiffness localized to the treated knee.

Note!

The product is only intended to be administered by authorized personnel in accordance with local legislation. Do not over screw the plug rod to prevent product leakage, and avoid stopper deformation that increases the extrusion force. The syringe, the needle and any unused material must be discarded directly after the treatment session. If the package is damaged, do not use. Before use, please refer to instruction.

Storage:

As indication on package, store under room temperature. Protect from sunlight and freezing.

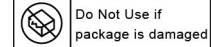
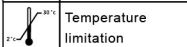
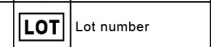
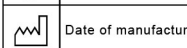
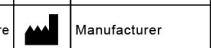
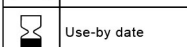
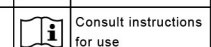
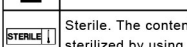
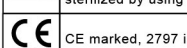
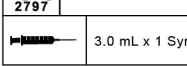
Shelf life: 3 years

SciVision Biotech Inc.

9, South 6th Rd., K.E.P.Z., Kaohsiung 806, Taiwan, R.O.C.
E-mail:service@scivision.com.tw

Distributed by **Maco Pharma S.A.S**
Rue Lorthiois, 59420 Mouvaux, France

SYMBOLS ON PACKAGING :

	Do not re-use		Do Not Use if package is damaged
	Temperature limitation		Lot number
	Date of manufacture		Manufacturer
	Use-by date		Consult instructions for use
	Sterile. The contents of the syringe have been sterilized by using moist heat.		
	CE marked, 2797 is No. of Notified Body.		
	3.0 mL x 1 Syringe		

EC REP
Obelis s.a.
Bd. General Wahis 53, 1030 Brussels, Belgium
Tél: +32 2732 5954
Fax: +32 2732 6003
E-mail: mail@obelis.net

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Distribuido por **Maco Pharma S.A.S**